# VERMEIREN

## Stephanie

INSTRUCTION MANUAL  
MODE D'EMPLOI  
GEBRUIKSAANWIJZING  
GEBRAUCHSANWEISUNG  
ISTRUZIONI PER L'USO  
MANUAL DE INSTRUCCIONES  
INSTRUKCJA OBSŁUGI

### **Instructions to the specialist dealer**

This instruction manual is part and parcel of the product and must accompany every product sold.

Version: D, 2016-08

All rights reserved, including translation.

No part of this manual may be reproduced in any form what so ever (print, photocopy, microfilm or any other process) without written permission of the publisher, or processed, duplicated or distributed by using electronic systems.

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### Elbow crutch Stephanie

#### 1. Technical details

**⚠ WARNING: Risk of unsafe settings - Use only the settings described in this manual.**

<b>Make</b>	Vermeiren	
<b>Address</b>	Vermeirenplein 1/15, B-2920 Kalmthout	
<b>Type</b>	Elbow crutch	
<b>Model</b>	Stephanie	
Width	155 mm	
Total height	940 mm – 1170 mm	
Height handgrip	745 mm – 975 mm	
Distance handgrip - elbow support	225 mm	
Weight	0,55 kg / piece	
Maximum user weight	max. <b>130 kg</b>	
We reserve the right to introduce technical changes. Measurement tolerance +/- 15 mm / 1,5 kg		

First of all we wish to thank you for the trust you placed in us by selecting a **VERMEIREN** product.

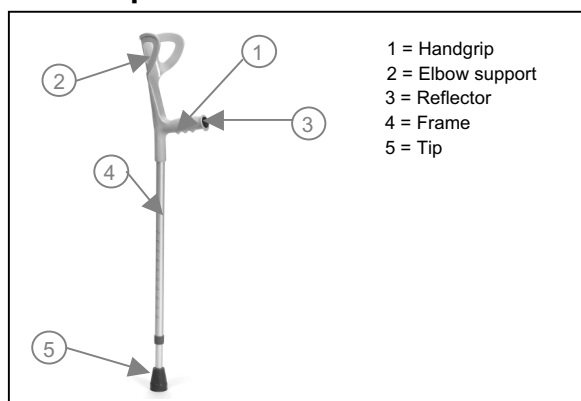
Before using the elbow crutches, read the instruction manual carefully: it will familiarise you with this product.

Refer to the Vermeiren website for the instructions for the care and safety of the Elbow crutch. Please remember that observance of our hints would keep your elbow crutches in an excellent condition of preservation and it will function perfectly even after years of use.

If you have any further questions, please consult your specialist dealer.

To find a service facility or specialist dealer near you, contact the nearest Vermeiren facility. A list of Vermeiren facilities can be found on the last page.

#### 2. Components

#### 3. Explanation of symbols

Maximum mass

Indoor and outdoor use

CE conformity

Read instruction manual before use

Observe the safety instructions

#### 4. Check after delivery

Unpack your product and verify if the delivery is complete. Following items should be included:

- 2 elbow crutches (ordered as set)
- 1 elbow crutch (ordered separate)
- Manual

Verify your product for transport damage. If you find any damages after delivery proceed as follows:

- contact the transporter
- have a list made of any problems
- contact your supplier

#### 5. Intended use

**⚠ WARNING: Risk of falling - Do not use the elbow crutch(es) if you have weak arm, wrist joint function or poor equilibrium.**

An elbow crutch gives the user support during walking. This support can be useful for fractures of the leg, rehabilitation after surgery, also for balance problems or pain when walking.

The elbow crutch can be used indoor and outdoor separate or as set on both sides.

Be extra careful that you do not slip when using on wet surfaces.

However, it cannot be used by persons with a visible weakened hand-/arm function, wrist joints or by persons with a balance problem.

#### 6. Height adjustment

**⚠ CAUTION: Risk of falling - Ensure that both crutches are placed on the same height.**

**⚠ CAUTION: Risk of falling - Check that the spring pins are good clicked back.**

Before using the crutch you must adjust these in the correct height.

Adjust the height of the crutch(es) according following instructions:

1. Press on the spring pin of the relevant crutch.
2. Adjust the crutch on the desired height.  
Ensure that the height of your wrist is at the same height of the handgrip when you leave your arms loose next your body. Ensure that your body is in an upright position.
3. Check that the spring pin is good clicked back.
4. Adjust the other crutch with the same instructions and on the same height (when used as a set)

## 7. Walking

**⚠ CAUTION: Risk of falling - Do not place the crutches too far to the front.**

Follow the advice of your doctor or physiotherapist how you must walk with the crutches. When you have no advice, you can use the instructions below:

- Always walk upright with the crutch(es) right next to your body.
- Put the crutch while walking never further than your front leg.
- Be careful with wet/slippery ground.
- Do not place the crutches in a curve while walking otherwise you can slip out.
- When you need just 1 crutch, place the crutch on the side of your unaffected leg. Place your affected leg forward simultaneously with the crutch and then your unaffected leg.
- When you need 2 crutches and you may not support on the affected leg, move the crutches to the front and then close with your unaffected leg. Be careful you do not place the crutches too far to the front.
- When you need 2 crutches and you may support on the affected leg, move the right crutch and left leg simultaneously to the front, and with the right leg the left crutch.

## 8. Walking on stairs

When there is a handrail available, you can use these as support and on the other side you can use the crutch. Ensure that the handrail is on the healthy side of your body, if possible.

Lean on the handrail and place the crutch on the first step. Place your unaffected leg on the first step and put your affected leg beside them.

## 9. Safety instructions

- Note that you use your elbow crutches on a good surface. (Not with too many gravel).
- Be extra careful on a wet / slippery ground.
- Before every usage, check if the elbow crutch is adjusted in the correct height.
- Check that the spring pins for the height adjustment are good clicked back.
- Do not place the crutches in a curve are too far from your body to avoid risk of falling.
- Do not use the elbow crutches as a toy.
- Do not hold the handgrips with wet hands. If you do so, you can lose your grip and lose your balance.
- Do not take to big steps with the elbow crutches.
- **The max. user weight of the product is 130 kg.**
- Risk of burns – Be careful when using in hot or cold environments (sunshine, extreme cold, saunas, etc.) for a sufficient amount of time and when touching - Surfaces can assume the environment temperatures.
- Take note of the instructions for care and service. The manufacturer is not liable for damage caused by improper servicing/care.

## 10. Maintenance

For the maintenance manual of the elbow crutches refer to the Vermeiren website: [www.vermeiren.com](http://www.vermeiren.com).

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## ENGLISH

### CONTRACTUAL WARRANTY

We offer 5 years of warranty on standard wheelchairs, lightweight wheelchairs 4 years. Electronic wheelchairs, tricycles and other products: 2 years (batteries 6 months) and multiposition wheelchairs 3 years. This warranty is limited to the replacement of defective or spare parts.

### APPLICATION CONDITIONS

In order to claim its warranty, part "B" of this card has to be given to your official Vermeiren dealer. The warranty is only valid when parts are replaced by Vermeiren in Belgium.

### EXCEPTIONS

This warranty is not valid in case of:

- damage due to incorrect usage of the wheelchair,
- damage during transport,
- involvement in an accident,
- a dismount, modification or repair carried outside of our company and/or official Vermeiren dealership,
- normal wear of the wheelchair,
- non-return of the warranty card

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- please return within 8 days of date of purchase or register your product at our website, <http://www.vermeiren.be/registration>

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---

## Notes

A series of horizontal dotted lines for writing notes.

---

## Notes

A series of horizontal dotted lines for writing notes.

## Belgium

### N.V. Vermeiren N.V.

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)

## France

### Vermeiren France S.A.

Z. I., 5, Rue d'Ennevelin  
F-59710 Avelin  
Tel: +33(0)3 28 55 07 98  
Fax: +33(0)3 20 90 28 89  
website: [www.vermeiren.fr](http://www.vermeiren.fr)  
e-mail: [info@vermeiren.fr](mailto:info@vermeiren.fr)

## Italy

### Vermeiren Italia

Viale delle Industrie 5  
I-20020 Arese MI  
Tel: +39 02 99 77 07  
Fax: +39 02 93 58 56 17  
website: [www.vermeiren.it](http://www.vermeiren.it)  
e-mail: [info@vermeiren.it](mailto:info@vermeiren.it)

## Poland

### Vermeiren Polska Sp. z o.o

ul. Łączna 1  
PL-55-100 Trzebnica  
Tel: +48(0)71 387 42 00  
Fax: +48(0)71 387 05 74  
website: [www.vermeiren.pl](http://www.vermeiren.pl)  
e-mail: [info@vermeiren.pl](mailto:info@vermeiren.pl)

## Czech Republic

### Vermeiren ČR S.R.O.

Nádražní 132  
702 00 Ostrava 1  
Tel: +420 596 133 923  
Fax: +420 596 121 976  
website: [www.vermeiren.cz](http://www.vermeiren.cz)  
e-mail: [info@vermeiren.cz](mailto:info@vermeiren.cz)

## Germany

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Wahlerstraße 12 a  
D-40472 Düsseldorf  
Tel: +49(0)211 94 27 90  
Fax: +49(0)211 65 36 00  
website: [www.vermeiren.de](http://www.vermeiren.de)  
e-mail: [info@vermeiren.de](mailto:info@vermeiren.de)

## Austria

### Vermeiren Austria GmbH

Schärddinger Strasse 4  
A-4061 Pasching  
Tel: +43(0)7229 64900  
Fax: +43(0)7229 64900-90  
website: [www.vermeiren.at](http://www.vermeiren.at)  
e-mail: [info@vermeiren.at](mailto:info@vermeiren.at)

## Switzerland

### Vermeiren Suisse S.A.

Eisenbahnstrasse 62  
3645 Gwatt (Thun)  
Tel: +41(0)33 335 14 75  
Fax: +41(0)33 335 14 67  
website: [www.vermeiren.ch](http://www.vermeiren.ch)  
e-mail: [info@vermeiren.ch](mailto:info@vermeiren.ch)

## Spain / Portugal

### Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5  
Sant Gregori Parc Industrial Edifici A  
17150 Sant Gregori (Girona)  
Tel: +34 972 42 84 33  
Fax: +34 972 40 50 54  
website: [www.vermeiren.es](http://www.vermeiren.es)  
e-mail: [info@vermeiren.es](mailto:info@vermeiren.es)

## Manufacturer

### Vermeiren GROUP

Vermeirenplein 1 / 15  
B-2920 Kalmthout  
Tel: +32(0)3 620 20 20  
Fax: +32(0)3 666 48 94  
website: [www.vermeiren.be](http://www.vermeiren.be)  
e-mail: [info@vermeiren.be](mailto:info@vermeiren.be)